



Mask Care and Instructions





Nose Clip



Adjustable Ear Straps

To adjust straps, untie knot and retie it to your desired length

Filter

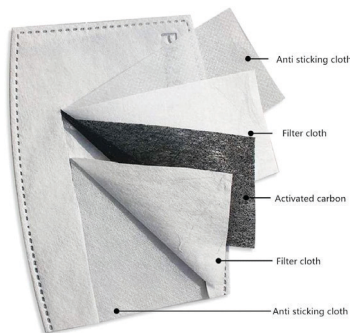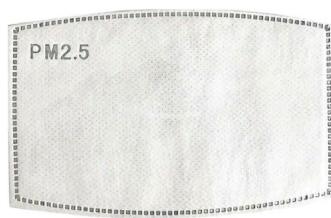

Before Washing Please Remove:

Nose Clip - Note there is an opening at the top towards the end where you can push the nose clip out.

Filter - is not washable, please replace or sanitize with UV light sanitizer. Replacements filters and UV Light box are sold on Amazon.

Washing

Adjust ear straps so they don't get caught on things in the wash or put mask in a mesh bag for washing.

Close Snap button at the button of the mask to keep it lasting longer and prevent rips.

Machine or hand wash with like colors in cold or warm water, especially on the first wash.

Thank you so much for your support, it goes towards more creative designs in the future. I hope you enjoy your new mask!

Filter Details

